


Full Face Mask – ‘FDA Approve’ (FM03)



Full-face mask (FM03)

Full face mask (FM03L/M/S) (FDA)

A. Structure: This face mask is composed of Faceplate, Connector, Cushion and Headgear.

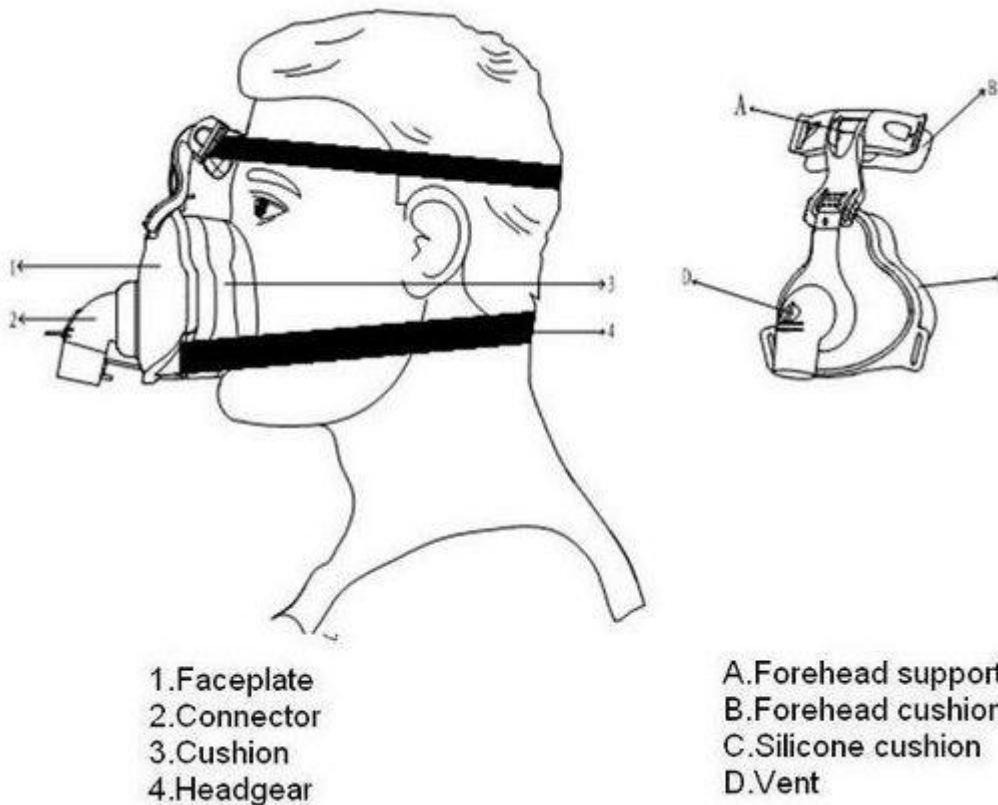

- 1.Faceplate
- 2.Connector
- 3.Cushion
- 4.Headgear

- A.Forehead support
- B.Forehead cushion
- C.Silicone cushion
- D.Vent

B. Intended Use: The Full Face mask is a ventilation device of CPAP or Bi-level therapy. It is used by medical care professional for oxygen transmission to patients.

C. Cleaning Instructions: This product can be repeatedly used after hand wash. The headgear should be cleaned weekly or when needed by hand.

D. Wearing Methods:

1. Connect the headgear to the mask. Position the mask over your face, and press the mask cushion gently against your face, pull the headgear over your head.



2. Adjust the top and the bottom straps so that the mask is sitting gently on your face.

3. Fasten the mask.



E. Features:

1. It's made of good, eco-friendly, medical silicone, plastic material
2. It's easy to wear and remove
3. It's well sealed to avoid oxygen leakage
4. It is light, safe, durable to use
5. It can well provide the oxygen to the patients who need

F. Storage Conditions:

Temperature: -5 °C-+50 °C.

Relative humidity \leq 85%, corrosive gas free, well-ventilated.