

Ms. WiZARD 230 Nasal Pillows *Mask and Headgear* - Fitting Guide

Fitting The Mask:

1. Ensure your nasal pillows mask is in the correct position. On the right headgear strap, the logo WiZard should be visible.
2. Insert the silicon pillows into the nostrils and pull the headgear over the head.
3. Adjust the side straps to a comfortable fit. The side straps should run between the ear and the eye.

4. Adjust the back straps to a comfortable fit. The pillows should sit snugly and soft in the nostrils.

5. If you prefer to fix the tube over your head, use the tube retainer buckle on top of the headgear.

6. Connect the swivel with the tubing and the flow generator (CPAP or bi-level machine) and turn on the airflow.

7. If necessary, adjust the headgear to eliminate any leaks without over-tightening the straps. If leaks still occur, change your silicon pillows size (three sizes are provided; Small, Medium and Large).