**EN****DeVilbiss Serenity® Mask Instruction Guide, 9352 Series**

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IMPORTANT SAFEGUARDS


Basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

- WARNING** Important safety information for hazards that might cause serious injury or death.
- CAUTION** Information for preventing damage to the product.
- NOTE** Information to which you should pay special attention.

PLEASE READ ALL INSTRUCTIONS BEFORE USING.

WARNING

- This CPAP mask should be used only with CPAP systems recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP system is turned on and operating properly. The vent hole(s) associated with the mask should never be blocked.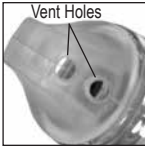

Explanation of the Warning: CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP unit is turned on and functioning properly, new air from the unit flushes the exhaled air out through the mask vent hole(s). However, when the CPAP unit is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP systems.

- CAUTION**—At low CPAP pressures, the flow through the exhalation port may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- NOTE**—At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate. This warning applies to most types of CPAP systems.
- If oxygen is used with the CPAP system, the oxygen flow must be turned off when the CPAP machine is not operating.
Explanation of the Warning: When the CPAP device is not in operation and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. This warning applies to most types of CPAP systems.



SAVE THESE INSTRUCTIONS

INTENDED USE (FOR SINGLE-PATIENT USE)

This product is intended to be used with nasal continuous positive airway pressure (CPAP), auto-adjusting CPAP and Bilevel systems for the treatment of Obstructive Sleep Apnea (OSA).

IMPORTANT PARTS OF YOUR DEVILBISS SERENITY MASK & HEADGEAR


- 1. 15mm Tubing with Swivel Adapter
- 2. Forehead Pads (silicone/gel/ ComfortTouch™ options)
- 3. Mask Body
- 4. Adjustable Vent
- 5. Mask Cushion (silicone/gel/ ComfortTouch options)
- 6. Forehead Rest
- 7. Headgear with Tubing Loop
- 8. Quick Release Strap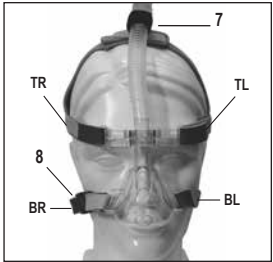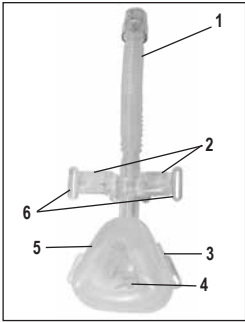

NOTE – Both mask and headgear have TR, TL, BR, BL imprinted designations.

SET-UP & OPERATION

Headgear Assembly

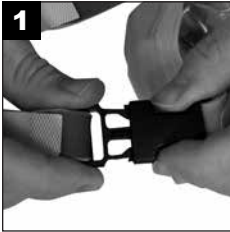To apply, unclasp quick release buckle.

Pull the assembled mask and headgear over your head.

Place the mask cushion over your nose, and ensure that the bottom straps are below your ears. Attach Quick Release buckle.

NOTE – Headgear crown strap can be adjusted with Velcro tabs after detaching loop/tubing tab. Reattach loop/tubing once adjusted.

The mask should be parallel to your face. Check the position of the mask on your face and adjust headgear tension starting with the Bottom Left (BL) and Bottom Right (BR) straps. You should be able to get your finger between the headgear strap and your face.


CAUTION – Do not overtighten your headgear! Excessive tension will cause discomfort during the night.

Adjust the Top Left (TL) and Top Right (TR) straps until the forehead pads rest comfortably against your face.


The position of the forehead rest can be adjusted to optimize comfort. To adjust the position of the forehead rest, simultaneously press the tabs inward on the forehead rest and pull the forehead rest upward away from the mask body.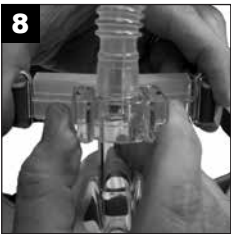


PUSH the forehead rest down in the best location and lock into place. Put the mask in position over your nose and ensure it is parallel to your face. If necessary, tighten top left and top right straps to eliminate leaks but still allowing your finger to go between face and headgear.

Circuit Connection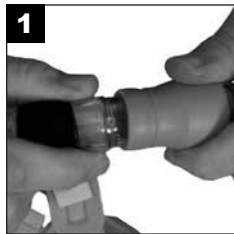

1
Attach CPAP tubing to 15mm tubing w/swivel adapter on Serenity mask

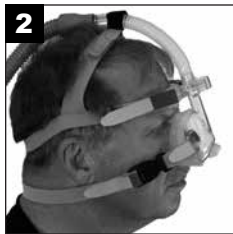2
Place mask on your face and turn on your CPAP device at the prescribed pressure to check for mask leaks.

Checking Mask Fit

1
To eliminate leaks on bottom portion of the mask, reposition mask and tighten bottom left and bottom right straps enough to eliminate leaks without over-tightening


2
To eliminate leaks and optimize comfort in the area around the bridge of the nose, either tighten top left and top right straps or adjust the headgear rest.


3
The 15 mm diameter tubing is attached to the mask and secured in the small loop on the top of the headgear. The position of this loop can be adjusted by removing and reapplying the Velcro strap.

Removing the Mask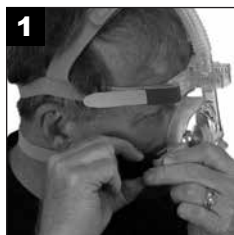

1
The Quick Release allows you to quickly and easily remove the headgear/mask assembly without affecting your headgear adjustments. If you need to remove your mask at any time during the night, unbuckle the Quick Release and lift the headgear/ mask off your head.

Adjustable Vent

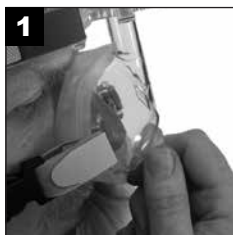1
Grasp adjustable vent and rotate to desired position to change the direction of airflow exiting exhaust port.

CAUTION – Do not overtighten your headgear! Excessive tension will cause discomfort during the night.

CARE AND CLEANING INSTRUCTIONS

Cleaning

Cleaning is recommended on a daily basis.

CAUTION: Do not use bleach, chlorine-, alcohol-, or aromatic-based cleaning solutions including scented oils. These solutions may cause hardening and reduce the life of the product. Do not place tubing in direct sunlight as it may harden and crack.

WARNING

CHOKING HAZARD. Contains small parts. Keep out of reach of children.

Disassemble the headgear, mask cushion, tubing, forehead pads, and adjustable vent. **NOTE**—Pull forehead pads off forehead rest tabs one side at a time.

Wash all of the parts, including headgear, with warm water and mild soap. Rinse thoroughly with warm water.


Allow parts to air dry.


NOTE—Do not towel dry parts; this could cause contamination.

Weekly Cleaning (if desired): The Serenity Mask is dishwasher safe if placed in top rack; small components should be secured. It is recommended that the mask be washed separately. Air dry before use.

REASSEMBLE MASK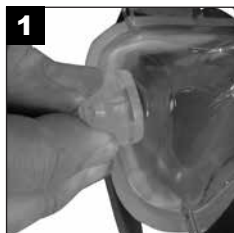

Fully insert adjustable vent into mask body from the outside. Ensure it is seated flat and rotates easily within the opening.


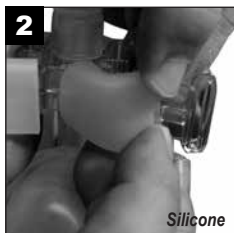
Silicone


Gel

Insert the tabs on the forehead rest into the openings on both sides of the forehead pad until secure.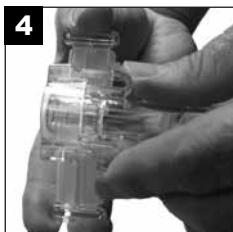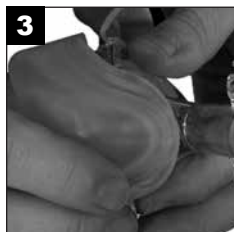


Replace mask cushion on mask body by first aligning the top alignment mark on cushion with alignment arrow on mask body. Push the top area of cushion onto mask body, then continue attaching along sides and bottom until cushion is completely secured. **NOTE – If necessary, adjust cushion position to eliminate wrinkles.**

Attach forehead rest to mask body.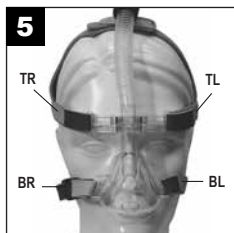

Reattach headgear to mask body. Hold the headgear so small loop is facing up and label is facing out. Insert the 2 top straps into the strap openings in the forehead rest (TL/TR). Insert the 2 bottom straps into the strap openings in mask body (BL/BR)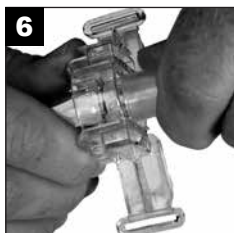

Attach 15 mm diameter tubing to mask body and insert tubing into small loop on top of headgear securing with strap.

NOTE – For optimum mask performance, it is recommended that you replace your mask at least one a year. See your DeVilbiss equipment provider for replacement information.

WARNING

This nasal CPAP mask is intended for single-patient use only. The mask may be cleaned and reused on the same person.

REPLACEMENT PARTS

	MODEL	DESCRIPTION		MODEL	DESCRIPTION
Serenity Complete Mask (Frame, Cushion, Headgear)	9352L	Large, Silicone	Cushion Replacement	9352L-601	Large, Silicone
	9352D	Medium, Silicone		9352D-601	Medium, Silicone
	9352S	Small, Silicone		9352S-601	Small, Silicone
	9352GL	Large, Gel		9352GL-601	Large, Gel
	9352G	Medium, Gel		9352G-601	Medium, Gel
	9352GS	Small, Gel		9352GS-601	Small, Gel
	9352LR	Large, ComfortTouch Silicone		9352LR-601	Large, ComfortTouch Silicone
	9352DR	Medium, ComfortTouch Silicone		9352DR-601	Medium, ComfortTouch Silicone
	9352SR	Small, ComfortTouch Silicone		9352SR-601	Small, ComfortTouch Silicone
	9352GLR	Large, ComfortTouch Gel		9352GLR-601	Large, ComfortTouch Gel
	9352GR	Medium, ComfortTouch Gel		9352GR-601	Medium, ComfortTouch Gel
	9352GSR	Small, ComfortTouch Gel		9352GSR-601	Small, ComfortTouch Gel
			Forehead Pads Replacement includes Adjustable Vent	9352D-604	Silicone
				9352G-604	Gel
				9352DR-604	ComfortTouch Silicone
				9352GR-604	ComfortTouch Gel
			Headgear	9352D-620	

Forehead Pads

Forehead pads are available in silicone or gel and can be interchanged. Refer to Reassemble Mask for replacement instructions.

DISPOSAL INSTRUCTIONS

Parts no longer used can be disposed of as normal household waste.

SPECIFICATIONS


Mask Physical Dead Space

MODEL	DESCRIPTION	VOLUME (ML)
9352S/9352SR	Silicone Small	140
9352D/9352DR	Silicone Medium	143
9352L/9352LR	Silicone Large	158
9352GS/9352GSR	Gel Small	145
9352G/9352GR	Gel Medium	143
9352GL/9352GLR	Gel Large	140

Mask Flow Resistance

FLOW RATE (L/MIN)	PRESSURE DROP (CMH ₂ O)
50	0.442
100	1.782

Pressure Flow Graph

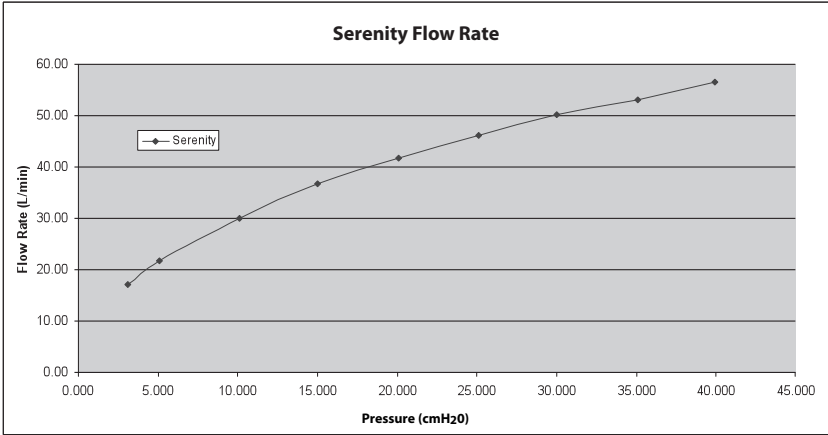Expected Service Life

- Mask Frame Components = 3 months
- Headgear = 6 months
- Mask Cushions = 1 month

WARRANTY

DeVilbiss Serenity Mask is warranted to be free from defective workmanship and material for a period of 90-days from date of purchase. Any defective part(s) will be repaired or replaced at DeVilbiss' option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your DeVilbiss Healthcare provider or DeVilbiss Healthcare Service Department for instructions:

U.S.A. 800-338-1988, 814-443-4881
Europe +49-621-178-98-230

NOTE– This warranty does not cover providing a loaner unit, compensating for costs incurred in rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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DeVilbiss® and DeVilbiss Serenity® are registered trademarks of DeVilbiss Healthcare. ComfortTouch™ is a trademark of DeVilbiss Healthcare.
@ 2014 DeVilbiss Healthcare LLC. 05.14 All Rights Reserved.
DeVilbiss Healthcare Pty. Limited
DeVilbiss Healthcare SAS
DeVilbiss Healthcare GmbH
DeVilbiss Healthcare LLC • 100 DeVilbiss Drive • Somerset, PA 15501 • USA