

Fitting Guide



Pixi Hint! If fitting the mask for the first time, we recommended the child be sitting in an upright position to ensure correct fit.



- Hold the mask gently against the child's face, checking that the nose is fully enclosed by the cushion and the mouth is clear.



- Pull the headgear over the head and move the middle strap down until it sits across the middle of the back of the head.



- Keeping your hand on the frame to maintain correct headgear position, bring the lower headgear strap behind the back of the neck.



- Attach the lower headgear strap by holding the tab and hooking the quick release clip onto the mask.



- If necessary, adjust the straps by removing the Velcro® and gently tightening the headgear straps until comfortable. Do not overtighten.
- For optimal results, adjust the straps starting at the bottom strap and work upwards.



- Connect one end of the air tubing to the swivel and the other end to the airflow device.
- Turn on the airflow device.

Tip



- When fully fitted, ensure the top strap is not overtightened and aligned with the top of the ears; the middle strap should sit across the back of the head aligned with the bottom of the ears.

 For details, refer to the **Pixi Pediatric Mask User Guide.**



Pixi™
PEDIATRIC MASK