

RESmart[®] CPAP

Maintenance Menu Setup

Entering the Menu: Press and hold the Pressure **Start/Stop** Button and **Ramp** Button while connecting the power cord into the AC inlet. After several seconds the Maintenance Menu will appear on display screen as below.

Treat	P
	8.0

Treatment Pressure

Treat P: The prescription pressure after ramp time. The setting range is 4-20cm H₂O. Adjustment step is 0.5cm H₂O. The Default setting is 8cm H₂O.

IMPORTANT! *Press the Ramp Button to enter change status and press +/- User Buttons to change the setting. Press Ramp Button again to confirm the setting. Press +/- User Buttons to access the next setting as below.*

<table> <tr><td>Treat</td><td>P</td></tr> <tr><td></td><td>8.0</td></tr> </table> <p>Treatment Pressure</p>	Treat	P		8.0	<table> <tr><td>Manual</td><td>P</td></tr> <tr><td></td><td>0.0</td></tr> </table> <p>Manual Pressure</p>	Manual	P		0.0	<table> <tr><td>Ramp</td><td></td></tr> <tr><td></td><td>15</td></tr> </table> <p>Ramp Timer</p>	Ramp			15
Treat	P													
	8.0													
Manual	P													
	0.0													
Ramp														
	15													
<table> <tr><td>Init</td><td>P</td></tr> <tr><td></td><td>4.0</td></tr> </table> <p>Ramp Starting Pressure</p>	Init	P		4.0	<table> <tr><td>Unit</td><td></td></tr> <tr><td></td><td>hPa</td></tr> </table> <p>Measurement Unit</p>	Unit			hPa	<table> <tr><td>Use</td><td>Time</td></tr> <tr><td></td><td>0</td></tr> </table> <p>Use Time</p>	Use	Time		0
Init	P													
	4.0													
Unit														
	hPa													
Use	Time													
	0													
<table border="1"> <tr><td>Save</td></tr> </table> <p>Saving</p>			Save											
Save														

IMPORTANT! *Any time you wish to exit the maintenance menu without change, press the Pressure Start/Stop Button and the display will go back to the system standby screen.*

IMPORTANT! *To make all changes active, press the +/- User Button to access the SAVE*

*setting and press **Ramp Button** to save all changes and exit the maintenance menu.*

- Manual P:** The adjustable range of treatment pressure within which the patient could readjust the treatment pressure to increase comfort by pressing the +/- user buttons on the device. The setting range is 0 to 2 hPa in 0.5 hPa increment.
- Ramp:** This setting is the ramp timer from starting pressure to prescription pressure. The device gives airflow at the relative lower pressure of Init P from start point of ramp time, and increases the output pressure to the prescription pressure in the last few minutes during ramp. Setting range is 0-60mins with the adjustment step of 5mins. Default setting is 15mins. The device reduces the output pressure and counts down to zero every time the Ramp key is pressed.
- Init P:** The starting pressure after blower is turned on. The setting range is 4hPa to Treatment Pressure (Prescription Pressure), and the Adjustment step is 0.5hPa. Default setting is 4hPa.
(Displayed when Ramp is non zero)
- Unit:** The measurement unit of the output pressure, selectable between hPa and cmH₂O.
- Use Time:** Total hours this RESmart device has been used will appear. This number is only for reference.
- Save and Exit:** If the settings are finished, please press the +/- user buttons to access the SAVE setting and press **Ramp** button to confirm all changes and exit the maintenance menu. If you want to cancel the settings, press **Start/Stop** button to roll back all your settings.