


CPAP Ventilator

Quick Start Guide

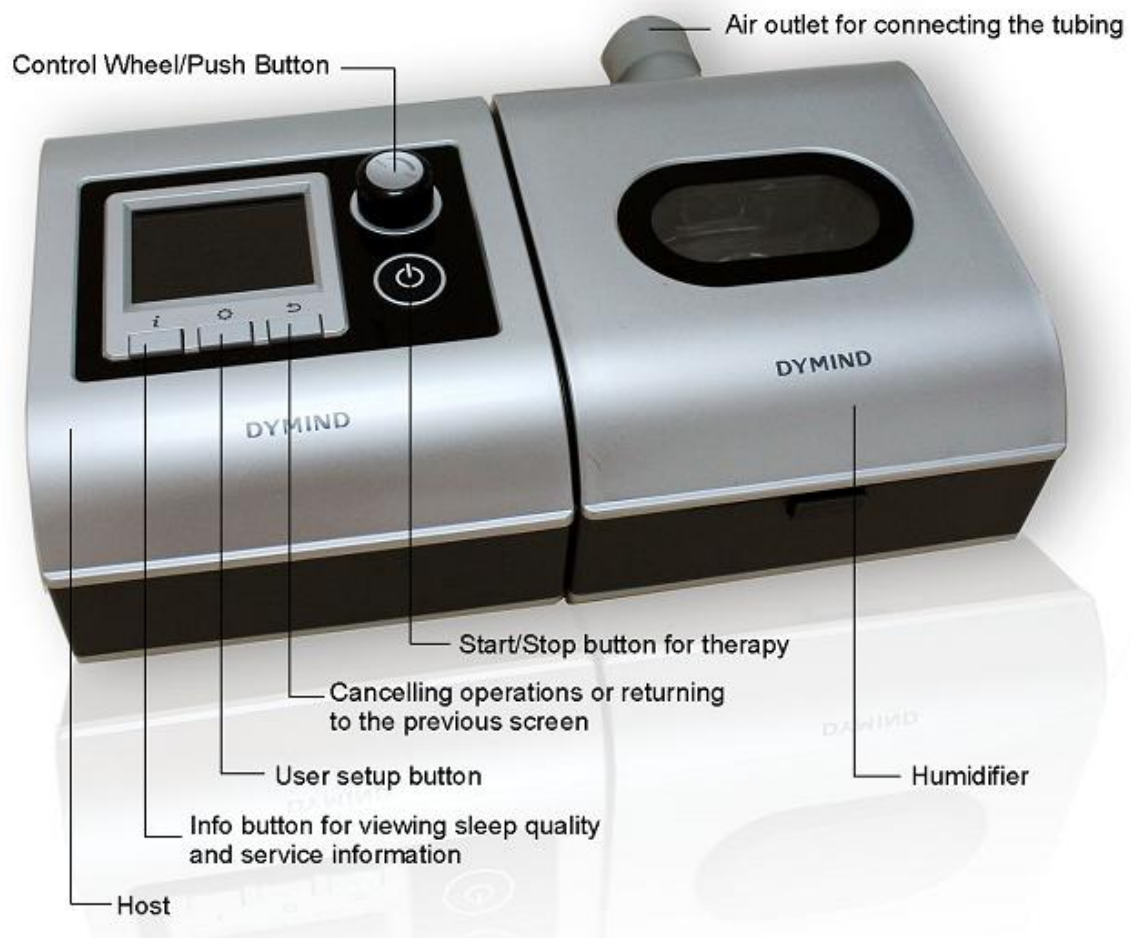

Read your entire **CPAP Ventilator User Manual** before use.

Installation



1. Connect the host to the heated humidifier. In alignment with the air channel and electrical connection of the humidifier, gently push the host so that the connecting clip of the host and that of the humidifier are locked to each other.



2. Take out the water tub and filled with purified water. Be sure not to exceed the highest water level.




3. Insert the water tub into the humidifier and gently press on the cap.



4. Connect one end of the tubing to the air outlet of the humidifier and the other end to the mask with the exhalation port.



5. Connect the DC power plug of the power adapter to the DC power interface on the back of the ventilator and connect the AC power plug to the AC power socket.

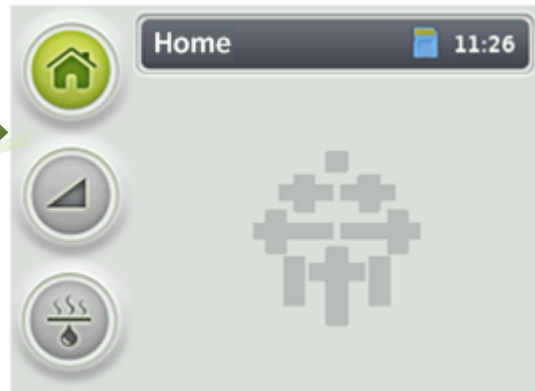

The **Home** screen appears on the display screen.


Adjusting to Therapy

If you are a first time user, you might need some time to get used to therapy. This is not unusual as it takes most patients 1~2 weeks to adjust to the most comfortable pressure. The following features will help you get used to therapy.

Setting the Ramp Time

1. On the **Home** screen, rotate the control wheel to the **RAMP** menu and press the control wheel to enter the **RAMP** screen
2. Rotate the control wheel to select the ramp time and press the control wheel to confirm the selection.

NOTE: When the ramp feature is enabled, the ventilator outputs an initial pressure and slowly increases the initial pressure to the therapeutic pressure during the predefined ramp time to help you fall asleep. When the ramp time ends, the ventilator automatically detects the patient's respiration conditions and adjusts pressure accordingly.

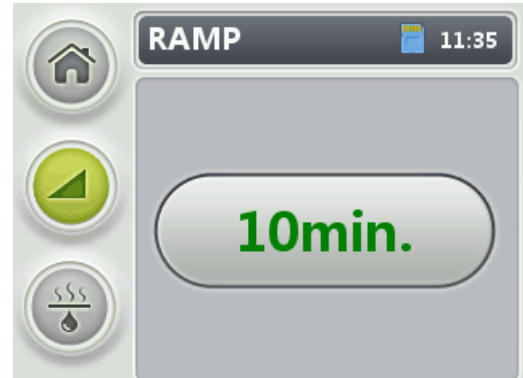

Changing the Humidity Level

The level of humidity can be set before or after cure. The value ranges from 1 to 6, or it can be set to **Off**.




1. Rotate the control wheel to the **Humidity Level** menu and press the control wheel to enter the **Humidity Level** screen.
2. Rotate the control wheel to select a humidity level and press the control wheel to confirm the selection




3. (When Humidity Level is not set to **Off**) Before treatment, press and held the control wheel for 3s. If the ventilator is not in **Warming Up** or **Cooling Down** state, the temperature and humidity levels of the humidifier are increased. If the ventilator is in **Warming Up** state, the temperature and humidity levels are reduced.

NOTE: During treatment, the ventilator will stop warming when the treatment is stopped.

Start Therapy

1. Put on the mask and tie the headbands and adjust them until you have a comfortable fit and there are no air leaks when you breathe.
2. Press the  button to start therapy.

Stop Therapy

1. Press the  button to stop therapy.
2. After using the ventilator, put off the mask and headbands and unplug the power cord to shut down the ventilator.

Daily Cleaning

1. Carefully clean the water tub, mask and air tubing with a mild detergent.
2. Rinse them with clean water; wipe the mask with a clean cloth to prevent stains.
3. Air dry, and avoid direct sunlight on the mask or place the mask on a radiator.
4. Wipe the outside of the ventilator with a cloth slightly dampened with water.

NOTE:

- Prevent ingress of water inside the ventilator during washing.
- Before using the ventilator, ensure that the enclosure is thoroughly dry and there is no moisture ingress inside the ventilator.

Parts Replacement Schedule

For your comfort and health, it is suggested that you replace your parts and supplies regularly.


Parts	Mask	Air tubing	Air Filter
Replacement Schedule	Every 3~6 months	Every year	Every 3~6 months



Shenzhen Dymind Biotechnology Co., Ltd.

2/F, Nanfeng Building B, Nanshan Yungu Innovation
Industrial Park, No.1183, Liuxian Blvd, Taoyuan Street,
Nanshan District, Shenzhen 518055, P.R.China

<http://www.dymind.com.cn/english>

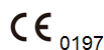

5 Bankside, Hanborough Business
Park, Witney Ox29 8LJ UK

P/N: 65020002AY (1.0)