

Fitting

Swift FX Bella



- Hold the pillows at your nose, ensuring the ResMed logo is above the pillows facing up. Pull one strap around and behind your ear.
- Ensure the strap lies flat and is not twisted.
- Repeat with the other strap.



- Adjust the mask fit by pulling the upper straps along the notches until a comfortable fit is achieved. Repeat with the lower straps. Do not overtighten.



- Connect the free end of the short tube assembly to the device air tubing.

Swift FX for Her



- Holding the pillows at your nose, pull the headgear over your head.
- Headgear should sit midway between the ears and eyes.



- Adjust the top strap first.
- Pull the strap through the buckle one notch at a time.
- Do not overtighten.



- Adjust the backstrap to fine-tune the fitting if necessary.
- Do not overtighten.
- The backstrap can be positioned over or under the hair.



- Swift FX pillows are designed to provide a natural fit.
- Minor adjustment can help position the pillows at the most comfortable angle in the nose.
- Connect the free end of the short tube assembly to the device air tubing.

Optional soft wraps

- Wrap each of the soft wraps around either side of the headgear, folding the top fastener over the lower one.
- The soft wraps should be positioned on the cheeks, with the fasteners on the outside.



S9™ Series
More. Comfort.



Combine a Swift FX for Her mask with ResMed's S9 AutoSet™ for Her therapy device for a comfortable, quiet, lifestyle-friendly system.

Swift™ FX Bella
NASAL PILLOWS SYSTEM

Less mask. More you.