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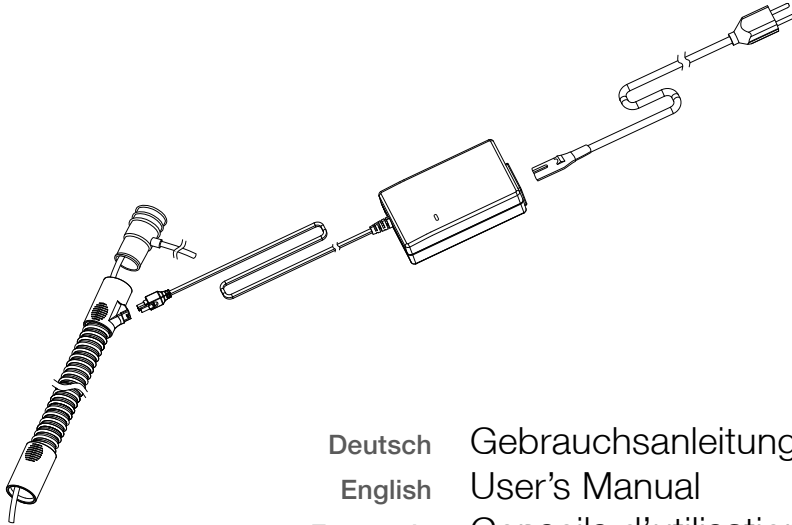
## TABLE OF CONTENTS

Symbols on the packaging.....	25
Warnings and precautions.....	26
Intended use.....	30
Description of the ComfortTube System .....	31
User's Manual .....	32
Cleaning .....	35
Frequently asked questions.....	37
Technical Data .....	38
Scope of delivery .....	40
Spare tube/-parts .....	41
Disposal.....	42
Manufacturers' guarantee .....	43

The ComfortTube System condensation control system is a tube system for adult patients who require respiratory support via positive pressure ventilation. The system consists of a heated therapy tube and an external power supply unit. The heating prevents condensation in the therapy tube. The ComfortTube System condensation control system can be used in sleep laboratories and at home.

**Before using the ComfortTube System, read this user's manual through carefully and completely.**

## SYMBOLS ON THE PACKAGING

Observe the warning and safety instructions in the user's manual.

Follow the user's manual

Product is latex-free

Product is DEHP-free

Product is only intended for use by a single patient

Do not use the product, if the package is damaged

Air humidity during operation, storage and transport

Permissible storage temperature

Manufacturer

Date of manufacture

Product can be used until

Article number

Batch designation

Manufacturer's CE mark

## WARNINGS AND PRECAUTIONS

- The ComfortTube System must only be used in combination with a humidifier, in order to prevent increased therapy air temperature.
- Please keep the user's manual for future use.
- Always touch the tube at the cuffs, when connecting or disconnecting the humidifier and the mask.
- Do not pull on the tube when disconnecting it from the humidifier.
- When using electrical devices, always adhere to local safety precautions.
- The ComfortTube System's power supply unit must not be immersed or brought into contact with water or a liquid.
- Position the power supply unit in such a way that there is no danger of tripping and the device cannot be pulled down.
- Do not use the ComfortTube System if the room temperature is higher than 30°C. Increased therapy air temperature can cause irritation of the airways.

- To avoid increased therapy air temperature, the ComfortTube System must not be used near sources of heat, such as lamps, electric blankets, etc. In addition, the therapy tube must not be covered with insulation sleeves (either made of fabric or plastic), cloths or other objects. Increased therapy air temperature can cause irritation of the airways.
- The ComfortTube System must only be used when the respiratory therapy device's blower is switched on.
- The tube must not be heated or sterilized.
- The tube may only be used by one patient.
- After washing, the tube must not be dried using a (microwave) oven, a hairdryer or other devices. Observe the cleaning instructions.
- The interior of the tube must not be cleaned with brushes or objects, as this can damage the tube.
- Do not disassemble the tube or damage it.
- Objects must not be inserted into the tube opening or allowed to fall into it.

- Materials or objects must not be placed on the tube, as this may damage the tube.
- Only spare parts (heated tube, power supply unit) recommended for the ComfortTube System by HOFFRICHTER may be used.
- The ComfortTube may only be operated using the power supply unit supplied with it. Products from other manufacturers must not be used.
- The power supply unit must not be cleaned using water or other liquids.
- The power supply unit must not be immersed or brought into contact with water or any other liquid.
- If the tube is damaged (holes, creases, tears, exposed heating cables, etc.) or no longer functional, do not try to repair it by yourself. Replace the tube immediately.
- Immediately replace the mains cable if the insulation or the plug is damaged.
- Dispose of the tube in accordance with local legislation.
- Label the tube so that it is not used by another patient before its final disposal.

- Immediately replace the tube if it is damaged or after the period defined by the physician or respiratory therapist has expired.
- The tube must be replaced after 12 months of use at the latest.

## INTENDED USE

The ComfortTube System is used to prevent condensation of the therapy air in the tube. It is used by patients receiving respiratory support via positive pressure ventilation.

The ComfortTube System is intended for non-invasive respiratory therapy in the home or sleep laboratory, and can also be used with an additional oxygen supply.

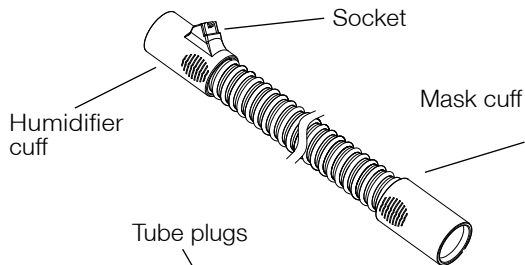
The ComfortTube System is intended to help patients experiencing problems with condensation in the tube due to the use of a humidifier. The ComfortTube System prevents condensation and thereby contributes to greater comfort during therapy.

The ComfortTube System may only be used by adult patients.


## DESCRIPTION OF THE COMFORTTUBE SYSTEM

### ComfortTube

heated therapy tube


Tube plugs
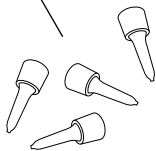
### Power supply unit

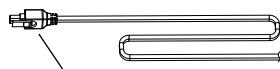
power supply inc. mains cable

Power supply plug

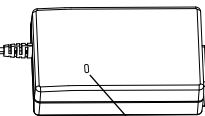
Mains plug adapter

Power supply


### Measuring tube adapter

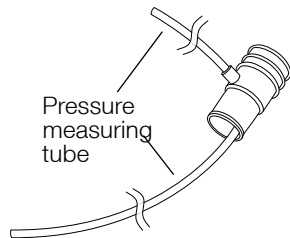Addition of a pressure measuring tube to the ComfortTube

Pressure measuring tube

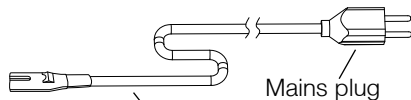Multi functional bag

Mains cable

Mains plug

## USER'S MANUAL

The ComfortTube is supplied with a measuring tube adapter, which is already connected to the ComfortTube. Use this combination if you are using a respiratory therapy device with a pressure measuring tube connection.

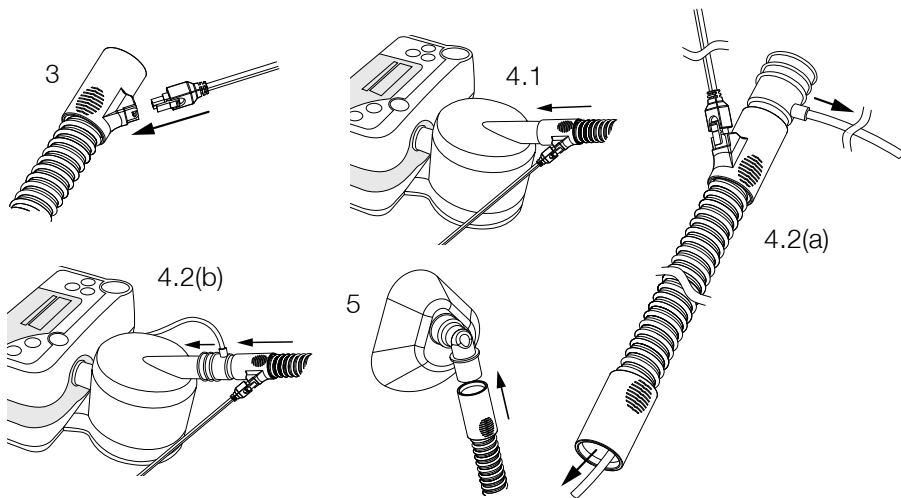
If you are using a respiratory therapy device without a pressure measuring tube connection, disconnect the measuring tube adapter from the humidifier cuff and pull the pressure measuring tube out of the therapy tube.

### How is it connected?

1. Remove the packaging from the therapy tube.
2. Before installation, check the tube, socket and power supply unit for damage.
3. Insert the power plug into the socket on the humidifier cuff.
4. Tube – humidifier connection
  - 4.1. Respiratory therapy device without pressure measuring tube connection  
Connect the humidifier cuff with the humidifier's air outlet.
  - 4.2. Respiratory therapy device with pressure measuring tube connection
    - (a) Pulling the pressure measuring tube allows you to change its position. Ensure that the pressure measuring tube extends far enough into the mask. The exact positioning of the pressure measuring tube depends on the mask. Please ask your specialist dealer about this.
    - (b) Connect the measuring tube adapter to the humidifier's air outlet. Attach the pressure measuring tube to the device's measuring tube connection
5. Connect the mask cuff to the mask.
6. Plug the power plug into a power socket and place the power supply unit onto the floor.


7. Use the LED indicator on the power supply to check if the power supply unit is working. When working properly, the LED lights up green.
8. Put the mask on and start respiratory therapy according to your physician or respiratory therapist's instructions, as well as the respiratory therapy device's user's manual.

## Disconnecting the power supply plug from the humidifier cuff

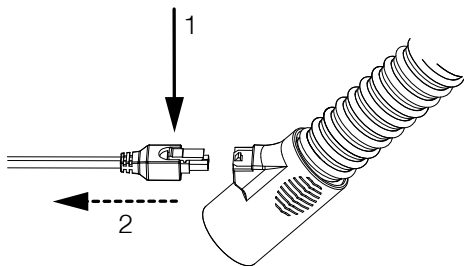Press down the lever (1) at the rear and carefully remove the plug (2) from the humidifier cuff.

## CLEANING

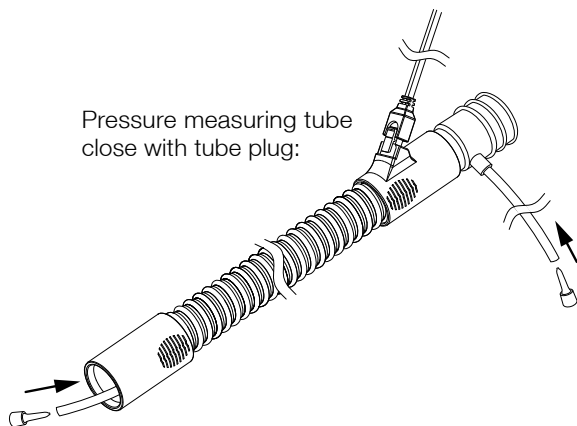
- Do not clean the power supply unit using water or other liquids.
- Do not use strong soap, detergents, solvents or alcohol to clean the tube.
- Do not clean the tube in the dishwasher.
- Do not use the ComfortTube System, if the room temperature higher than 30°C. Increased therapy air temperature can cause irritation of the airways.

Clean the tube daily. To do this, proceed as follows:

1. Remove the power supply plug from the humidifier cuff (see page 31).
2. When using the measuring tube adapter:  
Seal the pressure measuring tube using one tube plug at each end, so that moisture cannot enter the tube (see fig. on page 34). Residual moisture can lead to false pressure measurements or a delay in regulation. Both compromise optimal therapy.
3. Clean the tube in lukewarm, mild soapy water.
4. Rinse out the tube with clean water.
5. Gently tap on the humidifier cuff to remove the remaining water from the plug connection.
6. Hang up the tube with both cuffs pointing downwards.

7. Let the tube dry completely.
8. Remove both of the tube plugs.

## FREQUENTLY ASKED QUESTIONS

*Question: What will happen if I connect the mask cuff to the humidifier, and the humidifier cuff to the mask?*

**Answer:** The ComfortTube System works exactly as if the tube was connected correctly (mask cuff to mask and humidifier cuff to humidifier). However, for safety reasons and to ensure maximum comfort, connect the tube as described on page 10.

*Question: Can I use the tube without a power supply?*

**Answer:** Yes, the tube will then works like a standard tube for respiratory therapy devices. Respiratory air condensation can however form in the tube.

*Question: When do I have to replace the tube?*

**Answer:** The tube must not be used for longer than 12 months. If the tube is damaged or a leak or malfunction occurs, then the tube should be replaced immediately.

*Question: Which humidifier or flow generator can I use the ComfortTube System with?*

**Answer:** The cuffs of the ComfortTube and the adapter have the same diameter as standard tubes for respiratory therapy systems. Therefore, you can use the tube/adapter with all humidifiers that are intended for the connection of a standard tube.

## TECHNICAL DATA

ComfortTube (heated therapy tube)	
Output	max. 18 W
Power Supply	12 Volt (max. 1.25 A)
Length	1.83 m
Inner diameter	19 mm
Tube standards	ISO 5367:2000, ISO 8185:2007
Protection against water (protection class)	IPx3

Power supply unit	
Frequency	47 - 63 Hz
Power supply	100 V - 240 V
Electrical safety	EN 60601-1  B application part
Protection against water (protection class)	IPx1

## Operating conditions

Operating temperature	+ 5 °C ... + 30 °C
Relative humidity	5 % ... 95 %

## Storage- and transport conditions

Storage temperature	- 20 °C ... + 60 °C
Relative humidity	5 % ... 95 %
Storage conditions	The ComfortTube System should be transported and stored in the original packaging and protected from dust and exposure to direct sunlight.

## SCOPE OF DELIVERY

Scope of delivery	Article number
ComfortTube System consisting of: <ul style="list-style-type: none"><li>• 1 ComfortTube</li><li>• 1 measuring tube adapter</li><li>• 4 tube plugs</li><li>• 1 power supply unit (power supply inc. mains cable)</li></ul>	0000 3479
Multi functional bag	0000 2121
Mains plug adapter to be used with different AC connections	3210 0049
User's manual deu, eng, fra, nld	5000 0422
User's manual spa, por, ita	5000 0423
User's manual tur, ell, lit	5000 0424
User's manual pol, cze	5000 0425

## SPARE TUBE/-PARTS

The ComfortTube and the measuring tube adapter should not be used for longer than 12 months and should be replaced once this period is over.

The ComfortTube and the measuring tube adapter are intended exclusively for use with the ComfortTube System.

Should problems occur, ask your physician, respiratory therapist or nursing staff for support in replacing components.

Spare tube/-parts	Article number
ComfortTube inc. measuring tube adapter	0000 3480
Measuring tube adapter	0000 3481
Power supply unit (power supply inc. mains cable)	0000 3482

## DISPOSAL

The ComfortTube and the power supply unit must not be disposed of with the household waste. They must be disposed of at a communal collection point or public waste disposal authority.

The packaging, the measuring tube adapter, the tube plugs, and the required user's manual can be disposed of with the household waste.

## MANUFACTURERS' GUARANTEE

- The guarantee for the ComfortTube and measuring tube adapter is valid for 30 days, when used for max. 8 hours/day.
- The guarantee for the power supply unit is valid for 2 years, when used for 24 hours/day (at 100 - 240 V AC, 47 - 63 Hz) and normal charging current).

Warranty and guarantee expires, if:

- interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons who are not authorized by the manufacturer,
- other manufacturers' accessories and spare parts are used that have not been approved by the manufacturer for use the product,
- the product is used other than as described in the user's manual,
- wearing parts are worn,
- product damages are caused by force majeure (e.g. by lightning) or
- the hygiene and cleaning instructions described in the user's manual have not been complied with.

Statutory warranty rights remain unaffected by this.

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HOFFRICHTER GmbH