
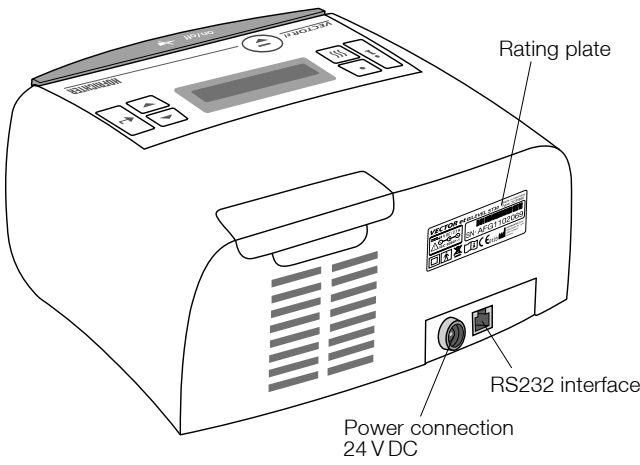



! This document does not replace the manufacturer's user's manual. Follow the user's manual.

HOFFRICHTER
Quality makes the Difference



VECTOR et brief instructions
for patients as of device software 2.140

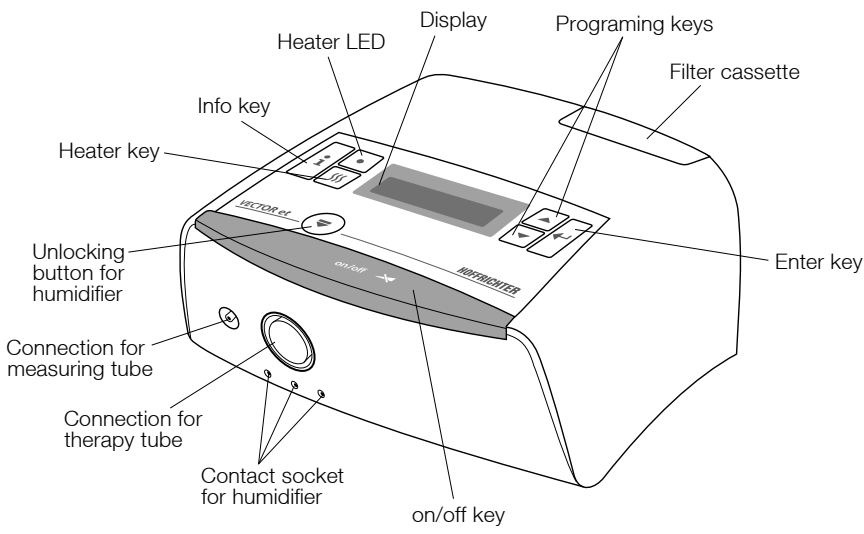

Insert SD card:

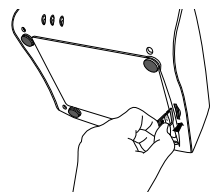

Remove SD card:

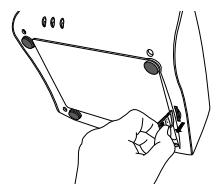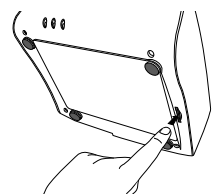


START/END THERAPY


To start or end the therapy, press the on/off key.

SETTING THE PARAMETERS

The following 3 keys are available to set parameters at the device:

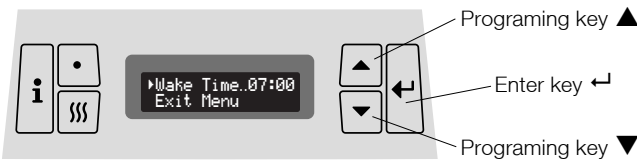

1. Press the enter key ↵ for approx. 1 second.
2. Switch to the next parameter with the programming key ▲ in following sequence:

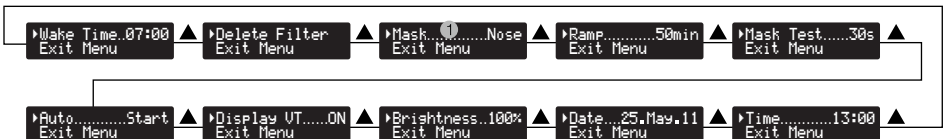

A flashing arrow ▶ shows the active parameter.

3. Press the enter key ↵ to change a parameter.
4. Adjust the desired value with the programming key ▲ and ▼.
5. Confirm the new value by pressing the enter key ↵.

① only VECTOR et AUTO

ALARM CLOCK ON/OFF

- Activate the alarm clock by using the programming key ▲. The bell symbol appears at the display.
- Using the programming key ▼ the alarm clock will be deactivated. The bell symbol disappears at the display.

SETTING THE WAKE-UP TIME

1. Press the enter key ↵ for approx. 1 second.
2. Select **Wake Time** by using the programming keys ▲ and ▼.
3. Confirm by pressing the enter key ↵.
4. Set the wake-up time by using the programming keys ▲ and ▼.
5. Confirm the wake-up time by pressing the enter key ↵.




INFO-MENU

Enter the Info-Menu by pressing the info key **i**. Here all adjusted parameters can be recalled. Changes of these parameters can only be done by your physician or authorized dealer.

REPLACING THE FILTER CASSETTE

If the filter is polluted or the display shows the message **Change Filter**, the filter cassette must be replaced.

1. To replace the filter cassette, pull it out of the device.
2. Replace filter cassette and filter frame cover.
3. Replace the filter cassette by a new one.
4. Insert the new filter cassette onto the filter frame cover. Make sure the upper part (slit) and the filter cassette (pin) are in the correct position.
5. Reset the filter counter. Press the enter key \leftarrow . Then select **Delete Filter** with the programming keys \blacktriangle and \blacktriangledown . Press the enter key \leftarrow twice.

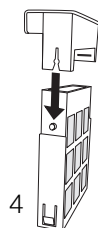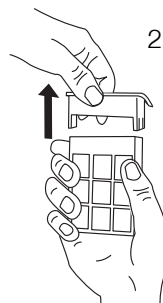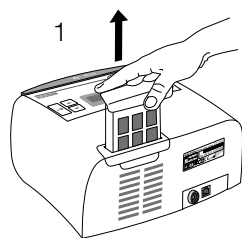


CAUTION


Never operate the device without filter cassette. Only and exclusively use original HOFFRICHTER filter cassettes.

HUMIDIFIER AquaDROP et (optional)

Connect the humidifier:

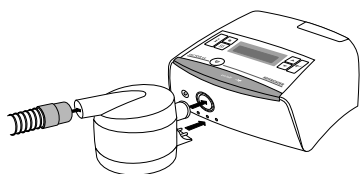

Detach the humidifier from the device:

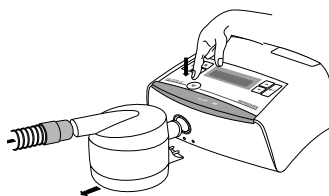

Setting heating level:

1. Press the heater key \lll until the heater menu is shown.
2. Press the enter key \leftarrow . The display of the heating level starts flashing.
3. Using the programming keys \blacktriangle and \blacktriangledown to set the desired heating level. Level 1 represents the lowest and level 5 the maximum heat output.
4. Confirm the setting using the enter key \leftarrow .



Press the heating key \lll to **switch on** the humidifier's heating. The display now shows the currently set heating level and the heater LED lights.

Press the heating key \lll again to **switch off** the humidifier's heating. The heater LED goes off. For more information please see AquaDROP et - user's manual.

CLEANING

DAILY CLEANING OF THE MASK

1. Separate mask and therapy tube.
2. Use mild soap water (don't use other additives) for cleaning, thoroughly rinse with clear water and let the mask dry at the air. Protect it from direct sunlight exposure.

WEEKLY CLEANING OF THE THERAPY TUBE

1. Before cleaning close the measuring tube using the stoppers.
2. Use mild soapy water for cleaning, thoroughly rinse with clear water, and let the tube dry at the air.
3. Remove the two stoppers from the measuring tube after cleaning.

IMPORTANT

If there is water remaining in the measuring tube pressure measuring may negatively affect the therapy.

WEEKLY CLEANING OF THE DEVICE

1. Disconnect the power plug from the socket.
2. Wipe the device with a cloth slightly moistened with soapy water. Then use a dry cloth to wipe the device dry.

CLEANING THE HEAD GEAR

Clean the head gear as required:

1. Disconnect the head gear from the mask.
2. Clean the head gear as described in the head gear manufacturer's user's manual.

