

Mirage Micro™



NASAL MASK

Mirage Activa™ LT


NASAL MASK

Mirage SoftGel™


NASAL MASK

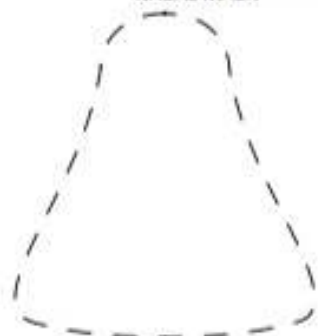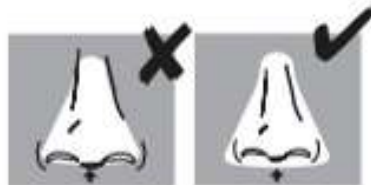


SMALL
(Micro & Activa LT Only)

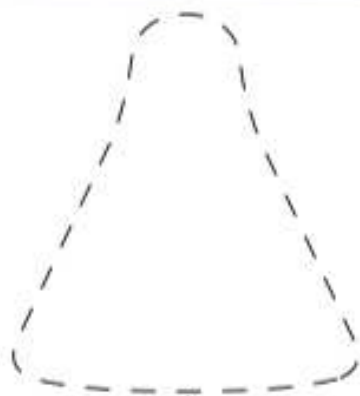


MEDIUM

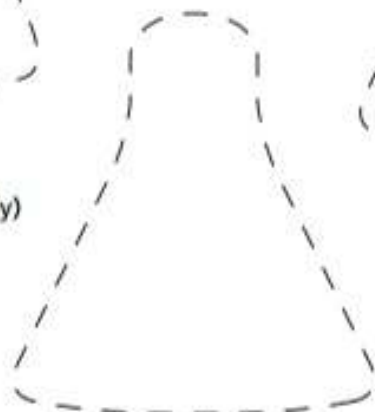

LARGE

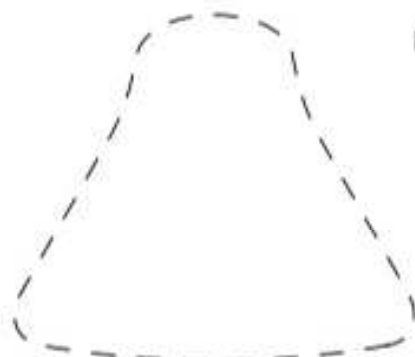

LARGE WIDE
(Micro & Activa LT Only)

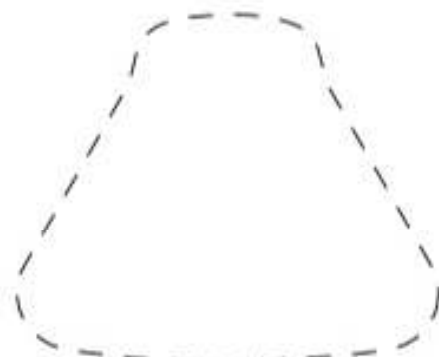

X-LARGE
(Micro Only)



How to use the fitting template:

1. Cut out the template along the dotted line
2. Cut out nasal holes
(hint: fold each nasal hole in half to cut)
3. Follow directions on template to determine appropriate cushion size

Actual Size: 5 1/2" (Width) x 7" (Height)

Ruler is for scaling purposes.

RESMED