



Barrel Cozy AirFit P10



- Place the Barrel Cozy over the nasal pillows.
- Gently pull the pillows through the holes, if the hole seems too small make a small slit in the edge of the hole. (The fabric will not fray)
- Wrap it around the barrel and attach the Velcro to secure it.
- Always make certain the vent of the mask is not covered by the Barrel Cozy.

This PAD A CHEEK Barrel Cozy is made of 100% polyester micro fiber and micro fleece. It is hand or machine washable and dryable (warm water and low heat)

Designed and Sewn in the USA

