



PAD A CHEEK[®]

Mask Liner Hybrid/Liberty



- Pull the nasal pillows of the mask completely through the holes in the [Mask Liner](#).
- Place the [Mask Liner](#) over the seal.
- Remove the lower side straps from your mask.
- Push the lower strap clips through the holes in the [Mask Liner](#) and clip them back into the mask.
- There is a hole cut for your mouth, you can customize it to your preference.
- **NOTE: If you need to cut the mouth hole larger, you can do this with sharp scissors. Trim a small amount at a time, it is easy to get the hole too large.**

The [PAD A CHEEK Mask Liner](#) is made from 100% polyester (micro fiber) and can be machine or hand washed with warm water and air or machine dried (low heat). If you have any questions, please contact me.

Designed and sewn in the USA

