

PAD A CHEEK[®]

Mask Liner Nasal N



Medium AirFit N20(M/L) and DreamWisp (M/L)

Small AirFit N20 Small, N10 all sizes, DreamWisp Small and Swift FX Nano

- Line up the top of the seal with the point in the mask liner with the triangle pad against the cushion of the mask.
- Put the cushion into the mask liner matching the hole with the nose opening in the mask.(Photo #2)
- Pull the leg over the side strap of the mask and attach with Velcro to the body of the mask.(Photo #3)
- Repeat step 3 on the other side. (Photo #4)
- Smooth the mask liner out to cover the seal (Photo #5).
- Re-attach the straps to the mask (Photo #6).

Make certain **no** exhalation vents are covered with the [PAD A CHEEK- Mask Liner](#).

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**



Photo # 1



Photo # 2

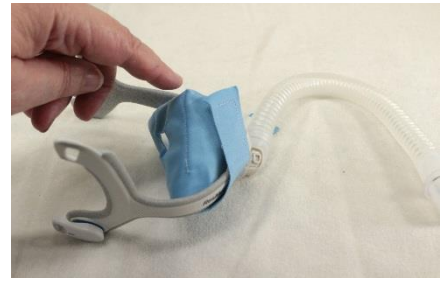


Photo # 3



Photo # 4



Photo # 5



Photo # 6

The [PAD A CHEEK Mask Liner](#) is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

Designed and Sewn in the USA