

PAD A CHEEK[®]



Mask Liner

Pico

- Lay out the mask liner (Photo #1).
- Place the wide part of the mask cushion into the bottom of the pocket of the mask. The triangle should be facing up (Photo #2).
- First, pull the flap that has the rough Velcro on it over the hose elbow connection (Photo #3).
- Then, pull the flap that has the soft Velcro on it over the hose elbow connection (Photo #4).
- Overlap the flaps against the mask, attach the Velcro together.
- Make sure the vents are not covered.


Make certain no exhalation vents are covered with the mask liner.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

#1



#2

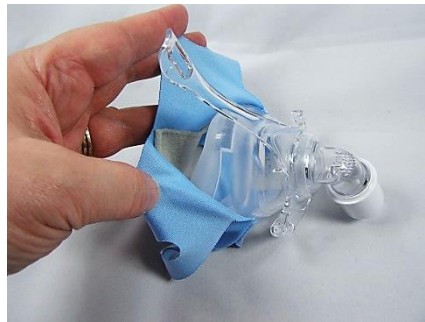

#3



#4



#5



The [PAD A CHEEK Pico Mask Liner](#) is made from 100% polyester (micro fiber and micro fleece), and can be machine or hand washed with warm water. If you have any questions, please contact me.

Designed and Sewn in the USA



PAD A CHEEK®



Forehead Pad Pico

(Includes Free additional padding piece. If needed, insert this into the forehead pad before installing on your mask)

- Remove one side strap from the mask forehead support.
- Lay the **Forehead Pad** onto the forehead part of the mask, with the label up and the cushion opening of your mask facing down on the table. (Photo #1)
- Push the strap through the hole in the end of the **Forehead Pad**, then thread the strap through the hole in the top of the mask support (Photo #2) and then out the same side of the **Forehead Pad**
- Secure with the Velcro. Repeat the same procedure for the other side. (Photo #3)
- Smooth the **Forehead Pad** until it lays flat.

PAD A CHEEK-Forehead pad is made from 100% polyester micro fleece, hand wash and air dry for best results.



Photo 1



Photo 2



Photo 3