

# PAD A CHEEK<sup>®</sup>

Strap Pad +



## Top of Head Pad

This fits the DreamWear series and the AirFit i series headgear

- Lay the **Top of Head Pad** under the hose connection tube with the pad in the center.
- You can place it with the fleece pad toward your head or away from your head.
- Wrap the Velcro tabs over the tubing and secure.
- Smooth down the **Top of Head Pad** so there are no wrinkles.

**Top of Head Pad** is 100% polyester microfiber and fleece and can be machine or hand washed.

