

AirFit™ N30i

AirFit™ P30i



Nasal cradle mask



Nasal pillows mask

Fitting guide




- Pull the frame and headgear over your head until the elbow is positioned on top and the headgear is behind your head.



- Place the cushion under your nose.
- For the AirFit P30i mask, align the pillows into your nostrils.



- Tighten or loosen the headgear until the cushion fits comfortably under your nose.

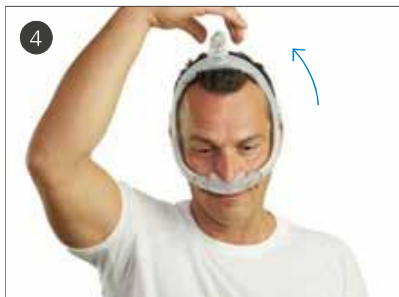

- Squeeze the side buttons on the elbow and detach from the frame.



- Connect the air tubing from your device to the elbow.



- Reattach the elbow to the frame.
- The mask is now ready to use.

Refer to the user guide for further instructions.



Additional options

- For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.
- Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.